

How to clean Touch Screen Guru medical grade & industrial grade monitors

Below are the approved disinfectants for use on Medical Grade Dual 4K/2K UHD Medical Monitors & Industrial Grade Full HD Touch Screen Monitors. The disinfectants have been validated on both housing and the touch glass. To prevent potential damage to the computers, please use the disinfectants listed below. Should you have any questions, please reach out to your Touch Screen Guru for additional information.

For cleaning best practices, please use a pre-moistened wipe with the disinfectant already applied, or spray the disinfectant on to a cleaning cloth and clean the unit. Spraying the disinfectant directly on to the unit may cause a drip hazard where the disinfectant will drip on to the floor if too much disinfectant is sprayed on to the unit.

Step 1 – Unplug the touch screen USB

Step 2 – Use a pre-moistened wipe with disinfectant already applied, or spray the disinfectant on to a non-abrasive cleaning cloth

Step 3 – Wipe the touch screen and monitor clean

Step 4 – Plug the touch screen USB in

Approved Cleaning Chemicals

SANI-CLOTH HB

SANI-CLOTH PLUS

SUPER SANI-CLOTH

Viraguard®

Ammonium Chloride 10%

Cavi Wipes

70% Isopropyl Alcohol

Clorox Disinfecting Wipes

Healthcare Hydrogen Peroxide Wipes

Alcohol Prep Pads

Super HDQL 10

Sodium Hypochlorite 6%